ADMECO™

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Lux

SURGICAL LIGHTS
ECLAIRAGES OPERATOIRES

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Why ADMECO-Lux?

ADMECO-Lux surgical lights were developed with one specific aim in mind – providing optimal operating theatre lights that did not interfere with air streams in laminar flow systems. The ADMECO-Lux lights were designed around these key criteria:

- maximal flow past surface of 450 cm²
- an open modular structure comprising individual projector heads
- projector head measurements of max. 8 – 10cm diameter
- projector head length of max. 20cm

Given these compact external measurements, temperature control needs to be extremely efficient with less than 10-watt residual energy.

The ADMECO-Lux design utilizes the radiant heat release in halogen lights, combining compact high-power halogen projectors with IR reflectors and IR permeable end caps to provide effective light intensity (measured at 1 m) of over 25,000 Lux per projector.

LED's (Light Emitting Diodes) would not be suitable for this design since their luminous power would have to be four times as strong to achieve the same light intensity as halogen with the same projector surface. High intensity discharge lamps, e.g., Xenon lamps, would need to have a far smaller wattage to avoid overheating.

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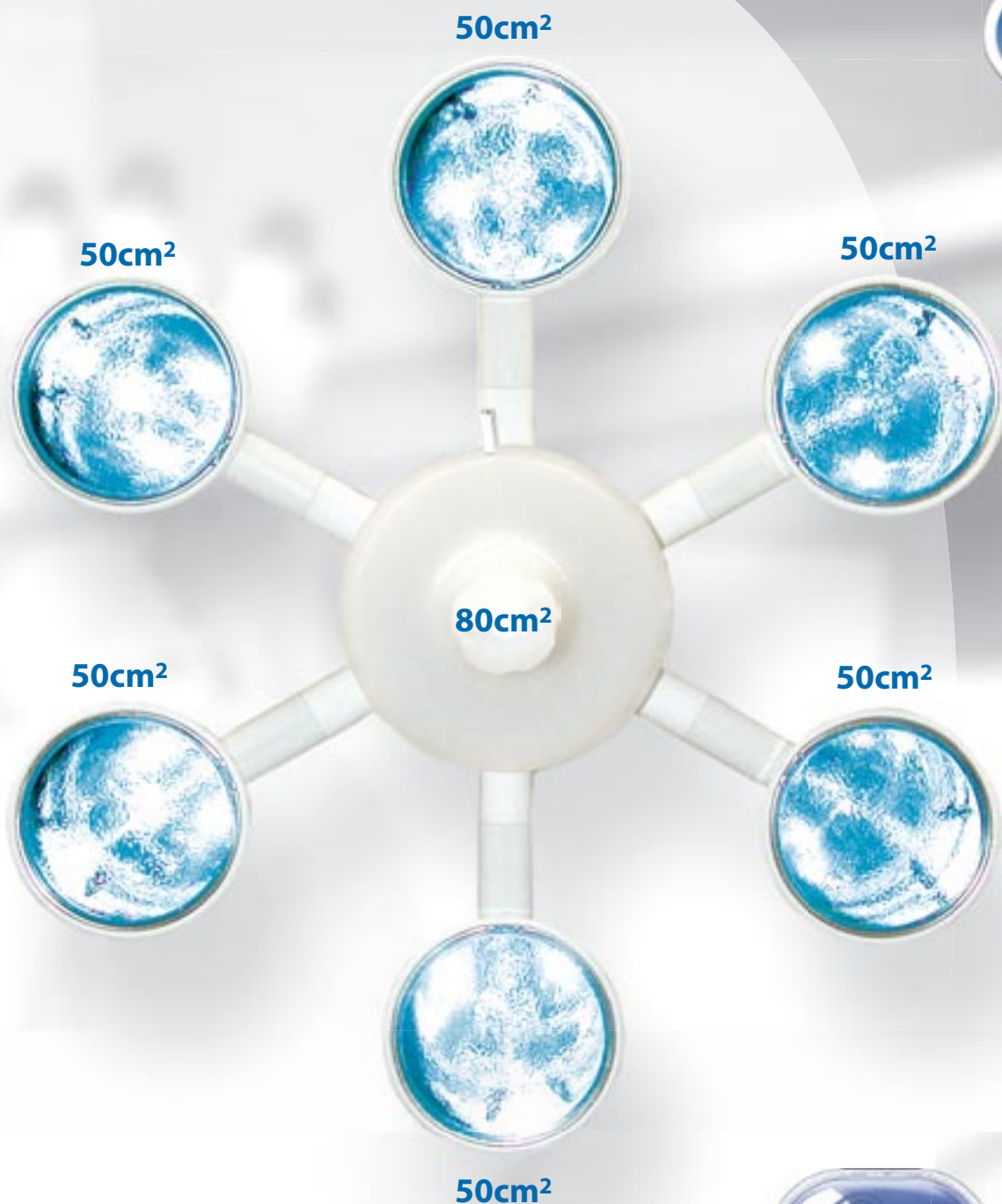
■ Measurements in Supplied OTs

The latest Swiss Guideline SWKI 99-3 shows just how great the difference is between ADMECO-Lux surgical lights and the lights of our competitors, with ADMECO-Lux lights rated as 100 to 1000 times more suitable for use in laminar flow systems (data on request).

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■ Full Light Intensity on the Wound Area

Since ADMECO-Lux surgical lights have such a compact shape (outer arc only 43cm), they can be positioned next to the surgeon's head, parallel to the angle of view, without the head intruding into the beam of light. As a result, the full light intensity is directed to the wound area. The correct angle of use allows the maximum of light to be reflected back towards the eyes. As a rule, a second surgical light is available and this can be positioned to disperse any residual shadow and expand the fully lit working area. A survey of more than one hundred diversely specialised surgeons using ADMECO-Lux found 85% were very satisfied with the performance of the surgical lights.

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Versatile and compact design

- Air stream optimized for laminar-flow compatibility
- Light, flexible and easy positioning
- Removable, sterilizable handles
- Easy replacement of Halogen/Xenon bulbs
- Uniform illumination accross working area
- Color correction for visual comfort
- Shadow-free in operation field
- Very sturdy construction
- Hermetically sealed

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High performance

- High Lux output/intensity
- Deep cavity penetration
- Sharply focused, high intensity light pattern
- Heat reflected upwards and transmitted through the IR-transmitter
- Dimmable lights
- High lux up to 4 x 150000 lux
- Color temperature 4300 K

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Wall mounting
suspension murale

Ceiling

- Individual lights 2 beams, 1-branch „SOLO“, not focusable
- Individual lights 3-6 beams, 1-branch „SOLO“, focusable
- Ceiling hanging surgical lights, 2-branches „DUO“, focusable
- Ceiling hanging surgical lights, max. 4-branches „MULTI“, focusable

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Integrated

- Individual lights 2 beams, not focusable
- Individual lights 3-6 beams, focusable

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Mobile

- Individual lights 1-2 beams, not focusable
- Individual lights 3-5 beams, focusable
- Optional with emergency power supply

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IR-Transmitter

Because of the infrared permeable hud of the lamp, most of the heat is exhausted upwards.

The result is a cooler working field for the surgeons.

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Diameter

The ADMECO-Lux is specially designed for laminar-flow outlets.

The very small diameter of only 8cm and the stream optimized shape of the light heads are disturbing the airflow at a minimum.

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Focus- / Positioning

The lamp can be positioned and focused fast and easily with the swivelling sterilizable handle.

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Easily maneuverable Aisance de mouvement

The rotation points A and B of the ADMECO-Lux OT-lights are close to the centre. This creates an extreme mobility almost comparable with a ball-joint.

With fingertip light manipulation ADMECO-Lux lamps can be positioned in the appropriate angle.

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Enlightening Design

- ***Innovative***
- ***High Quality***
- ***High Performance***

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
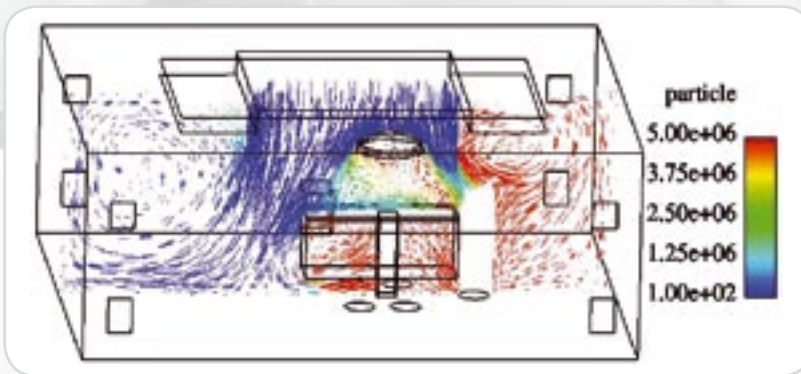
■ “The standard case is from a real situation except that persons around the table are represented by the dummies. The 6-lamp light and the big light are all placed in the same position, the big light is a dish-shaped light with a diameter of about 0.9m (CFD* 1), the 6-lamp light is made up of six 0.08m-diameter small lamps (CFD 2).”

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CFD 1: standard large dish shaped lamp
large coupole standard

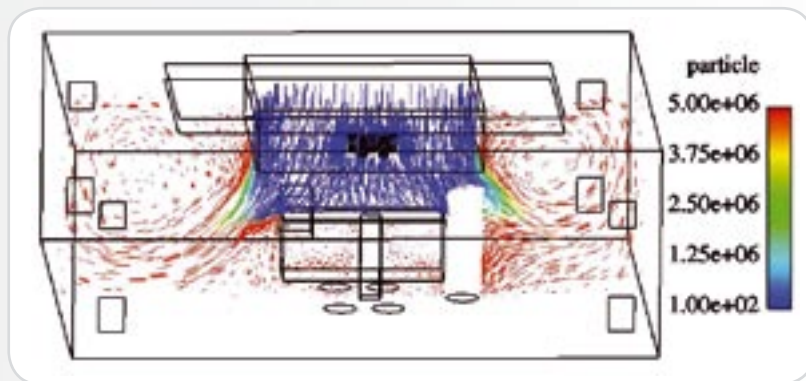CFD 2: 6-lamp light (ADMECO 6SI)
éclairage 6 projecteurs (ADMECO 6SI)

■ “The large lamp will disturb the flow field so much that the particle concentration above the table is seriously affected. CFD 1 shows the airflow vectors in the center plane of the table. It is obvious that this large lamp is not suitable to be placed between the air supply inlet and the operating table.”

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* CFD = Computational Fluid Dynamics

Surgical light with color video system

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The system includes:

- Color video camera integrated into a casing which can be attached to an ADMECO-Lux surgical light or a separate arm. (Lux models 3SI, 4SI and 5SI)
- Control unit incl. interference filter

Color video camera

The color video system designed to accompany the ADMECO-Lux surgical lights needs no additional, visible cables. The video camera is built into a casing which is then fastened to the ADMECO-Lux surgical light.

Control unit

The control unit of the color video camera allows you to adjust the iris, zoom, focus, white balance and rotation of the camera.

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Standard for all models:

- In - depth illumination > 0.9 (according to British Standard 4533 Section 103.2 1986)
- Average lifetime of bulbs at max. voltage 22.8V: 700 hours
- Each light head is equipped with 2 O-rings as a water- and dust-tight seal
- Bulbs: Special Halogen/Xenon bulb 50 Watt, 22.8 V with integrated dichroic (cool light) reflector or Halogen bulb 30 Watt, 13.8 Volt with integrated dichroic (cool light) reflector
- Each light head is equipped with 2 lenses, an interferencial and a color correction filter
- Ceiling support tube diameter 50 - 95mm to optimize the laminar-flow surface

(Right to change specifications reserved)

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We deliver ADMECO-Lux surgical lights to satisfied customers all over the world:

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ADMECO™

Lux

**Compact surgical light systems,
specially optimized for laminar-flow ceilings**

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Air & Climate, ETH Zürich: Airborne particle concentration control for an operating room:

"A large lamp will disturb the flow field so much that the particle concentration above the table is seriously affected. It is obvious that a large lamp is not suitable to be placed between an air supply inlet and an operating table":

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